


# ENAMEL PLUS TEMP

# Temp

for packages see section Laboratory pages 52-53



Fluorescent colors

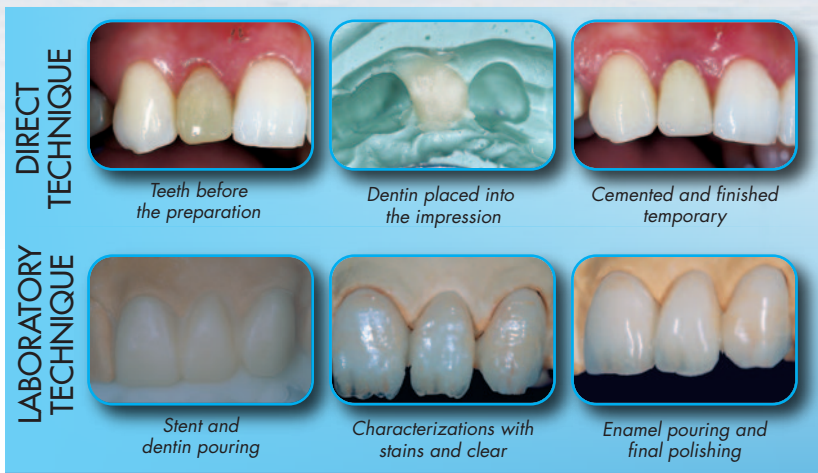

## Temporary resin for esthetic temporary crown and bridge

Very interesting material for opacity and fluorescence of dentin bodies and for translucency of enamel bodies that allows to put in evidence the typical contrast between these two elements in the natural tooth. Enamel bodies also allow to obtain a natural polishing in temporary crowns, similar to the definitive ceramic elements. It can be used both in laboratory with the help of silicone stent or of vacuum forming matrix with the direct technique using a silicone impression or a preformed crown.

## MOCK UP KIT

For temporary veneers



### TEMPORARY IN 5 MINUTES:

- impression with silicone and clear tray 45 sec.
- impression setting 1 min. 30 sec.
- veneer preparation
- Flow HRI application in stent 30 sec.
- light curing 60 sec.
- final light curing with Ena Seal 60 sec.

Ref	Description	Q.ty
TEMPSMU	<b>MOCK UP KIT</b>	
	Temp Silic Transparent Silicone for dentist	2x50 ml
	Auto Mixing Tips	8 pcs
	Anterior Transparent Tray	2 pcs
	Full Arch Upper Transparent Tray	2 pcs
	Full Arch Lower Transparent Tray	2 pcs
	Ena HRI Flow	3x2 g
	Black tips ø 1,2 mm	3 pcs
	Ena Seal	2,5 ml

Ref	Description	Q.ty
TS1	Transparent Silicone for dentist	50 ml
TS1-5	Transparent Silicone for dentist	5x50 ml
TSPA	Anterior Transparent Tray	25 pcs
TSPL	Full Arch Lower Transparent Tray	25 pcs
TSPS	Full Arch Upper Transparent Tray	25 pcs