

ENAMEL PLUS TEMP



AESTHETIC RESIN TEMPORARY



ENAMEL  TEMP
plus

HAND-MADE PROCEDURE



1

WAX-UP



Wax FLEX-E-Z Ivory

DENTINE POURING

2

6 universal dentine*
UD0, UD1 (A1), UD2 (A2),
UD3 (A3), UD3,5 (A3,5),
UD4 (A4)

*following colours also available on request:
B1, B2, B3, B4, C2, D3

STENT TECHNIQUE

Cold curing Liquid ref. TEMPL
Hot curing Liquid ref. TEMPLH



FLASK TECHNIQUE

Hot curing Liquid
under pressure for flask
ref. TEMLPRESS



ENAMEL CUTS

3



Images courtesy
D^r. Roberto Rossi - Md^t. Rondoni

OPALESCE & CHARACTERIZATION

4




3 opalescent enamels Clear, Gray, Blue

1 intensive enamel White

Hot curing Liquid TEMPLC

ENAMEL POURING

5

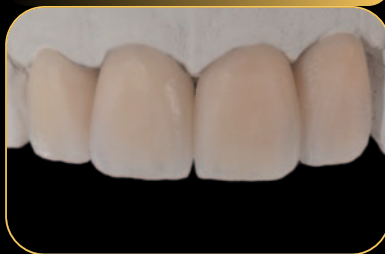

3 universal enamels

UE1 (old tooth)

UE2 (adult tooth)

UE3 (young tooth)

Cold, hot & under pressure Liquid

ENAMEL TEMP plus

Biocompatible*

Non-Toxic*

Non-allergic*


Antiplaque**

Fluorescent



POLISHING

6

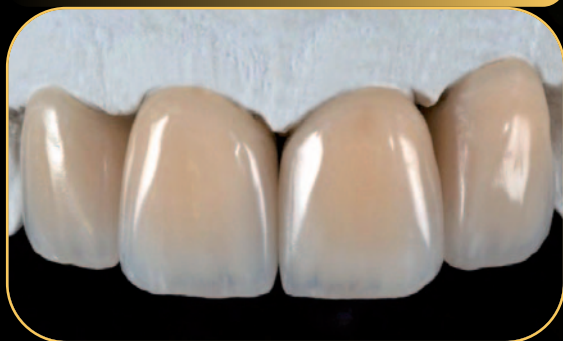

TEMPPOL liquid or TEMP99 Stick;

COSSHINYSHP BRUSH

HIGH PHYSICAL DATAS

Hand-made procedure

Water solubility:

1,1 $\mu\text{g}/\text{mm}^3$

Water absorption:

21.2 $\mu\text{g}/\text{mm}^3$

Hardness Vickers:

140 MPa

Modulus of elasticity:

2300 MPa

Residual monomer:

0,8 %

Flexural strength:

60-75 MPa

Material very interesting for opacity of dentine bodies and for translucency of enamel bodies that allow to put in evidence the typical contrast between these two elements in the natural tooth. Enamel bodies also allow to obtain a natural polishing in temporary crowns, similar to the definitive ceramic elements.

TEMPORARY WITH PAPILLA



Pink Temp Resin

ideal on implants & for partial dentures

* Cytotoxicity test with direct cell contact ASTM F 813-83 and with Agar diffusion at ASTM F895-84 with extraction of tests Irritation test and allergic sensitization by Buehler, OECD 406-92 and DIR CEE 93/21

ENAMEL TEMP plus



INTRODUCTORY KIT

TEMP1N INTRODUCTORY KIT AESTHETIC TEMPORARY RESIN CONTAINS:

4x50g new fluorescent dentines Enamel plus Temp:
UD2 (A2), UD3 (A3), UD3,5 (A3,5), UD4 (A4)
1x50g universal enamel Enamel plus Temp UE2
1x50g opalescent enamel Enamel plus Temp Blue OB
100ml self curing liquid Enamel plus Temp

TEMP1N-OF INTRODUCTORY KIT OFFER:

1 INTRODUCTORY KIT TEMP1N
+ 6x10g more fluorescent colors Enamel plus Temp
Dentine: UD1 (A1)
Enamels: UE1 - UE3 - Clear - White - Grey
TEMP99 Temp Stick Micerium Polishing Paste
Brush HP



LIQUID

TEMPL

Self curing liquid 100 ml for stent technique

TEMPLH

Hot curing liquid 100 ml for stent technique

TEMPLPRESS

Hot curing liquid 100 ml for flask technique

TEMPLC

Light curing liquid 10 ml for opalescence and characterization

Micerium S.p.A.
Via G. Marconi, 83
16036 Avegno (GE) Italy
Tel. +39 0185 7887 880 • www.micerium.com

File:TEMP EN v4_2023-10

