

# ENAMEL PLUS HRi FLOW<sup>HF</sup>

## TEMPORARY IN 5 MINUTES!

Temp Silic Mock up is an effective communication tool between the dentist, patient, and dental laboratory technician as allows to realize mock up with Enamel plus HRi Flow, a highly filled flowable composite with aesthetic features and ideal physical properties.

### OPERATING & TECHNOLOGICAL ADVANTAGES

- impression with silicone and clear tray **45 sec.**
- impression setting **1 min. 30 sec.**
- veneer preparation
- Flow HRi application in stent **30 sec.**
- light curing **60 sec.**
- final light curing with Ena Bond Seal **60 sec.**

### ENAMEL PLUS HRi FLOW<sup>HF</sup>

- Highly Filled
- Highly Fluorescent
- Higher Viscosity
- Higher Elasticity
- Higher Radiopacity
- High Physical Properties
- No bubbles



Initial case




Final result



### PHYSICAL CHARACTERISTICS

	FILLER (Weight)%	VICKERS HARDNESS MPa	YOUNG MODULUS MPa	FLEXURAL STRENGTH MPa	COMPRESSIVE STRENGTH MPa	RADIOPACITY
HRi Flow HF	77	770	13485	148	394	210% AI
Standard Flow	55	350	6000	120	350	30% AI

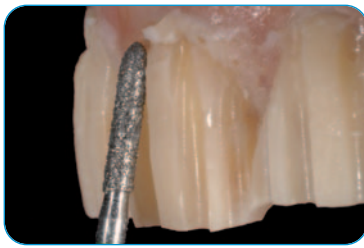


Veneer preparation on the mock up




Prepared teeth




Wax up on the model

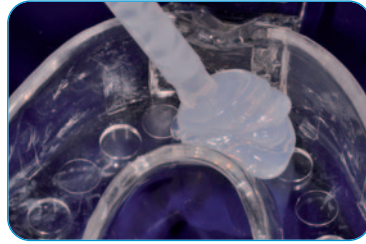

Transparent silicone & tray\*

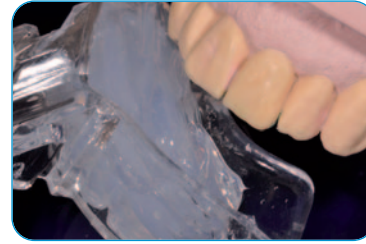


Silicone impression of wax up  
(setting 1 min. 30 sec)

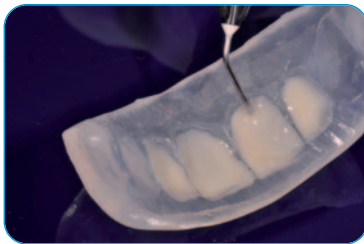


After removing wax up,  
Flow HRi\* application in silicone  
and curing on model (60 sec.)



Light curing  
bonding in  
temporary veneer

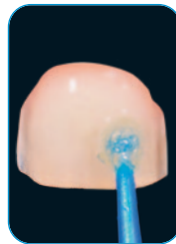

Ena Bond Seal on  
tooth surface

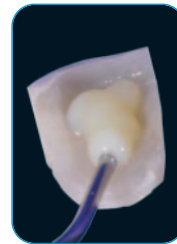

Ena Soft Flow\* in  
temporary  
veneer



Temporary veneers luted  
with Ena Soft Flow\*  
Final light curing with Ena Bond Seal



Ceramic Veneers



Final result after luting with light  
curing flowable composite  
(like Ena Cem HV)

#### KIT MOCK UP - FOR TEMPORARY VENEERS

Ref	Description	Q.ty
TEMPSMU	KIT MOCK UP	
	Temp Silic Transparent Silicone 70 shore	2x50 ml
	Auto Mixing Tips	8 pcs
	Anterior Transparent Tray	2 pcs
	Full Arch Upper Transparent Tray	2 pcs
	Full Arch Lower Transparent Tray	2 pcs
	Enamel plus HRi Flow	3x2 g
	Black tips ø 1,2 mm	3 pcs
	Ena Bond Seal	2,5 ml
	Ena Soft Flow	1 g
TS1	Temp Silic Transparent Silicone 70 shore	50 ml
TS1-5	Temp Silic Transparent Silicone 70 shore	5x50 ml
TSPA	Anterior Transparent Tray	25 pcs
TSPL	Full Arch Lower Transparent Tray	25 pcs
TSPS	Full Arch Upper Transparent Tray	25 pcs



Micerium S.p.A. Via G. Marconi, 83 - 16036 Avegno (GE) Italy  
Tel. (+39) 0185 7887 880 • hfo@micerium.it • www.micerium.com

\*For Enamel plus HRi Flow, Ena Soft Flow, Transparent Tray see the instructions for use

