



## TEMP SILIC

### **(I) Ritardante 5%**

Liquido ritardante da utilizzarsi per aumentare i tempi di lavorazione del silicone Temp Silic.

Aggiungere 1 goccia di ritardante (usando il contagocce del tappo del bottiglino stesso) per ogni coppia di misurini (A+B) di siliconi.

Es. se occorre preparare una quantità di prodotto pari a 5 misurini di A + 5 misurini di B, occorrerà aggiungere 5 gocce di ritardante.

### **(GB) 5% Retarding**

Retarding liquid to be used in order to increase the work time of Temp Silic silicone.

Add 1 retarding drop (using the dropper in the bottle cap) for each pair (A+B) of measuring spoon of silicones.

Ex. If you need a product quantity of 5 spoon of silicon A and 5 spoon of silicon B, you have to add 5 retarding drops.

MICERIUM S.p.A. Via G. Marconi 83 - 16030 - Avegno (GE) Italy - <http://www.micerium.it>

Telefax: +39-0185-7887970 - Tel. +39-0185-77887870 - e-mail: [micerium@micerium.it](mailto:micerium@micerium.it)

International direct line: Tel. +39-0185-77887880 - e-mail: [hfo@micerium.it](mailto:hfo@micerium.it)